


FAST-FIX[◇] 360 Meniscal Repair System

All-Inside Meniscal Repair

Knee Series Technique Guide
as described by:

Charles H. Brown, Jr., MD

Nicholas Sgaglione, MD

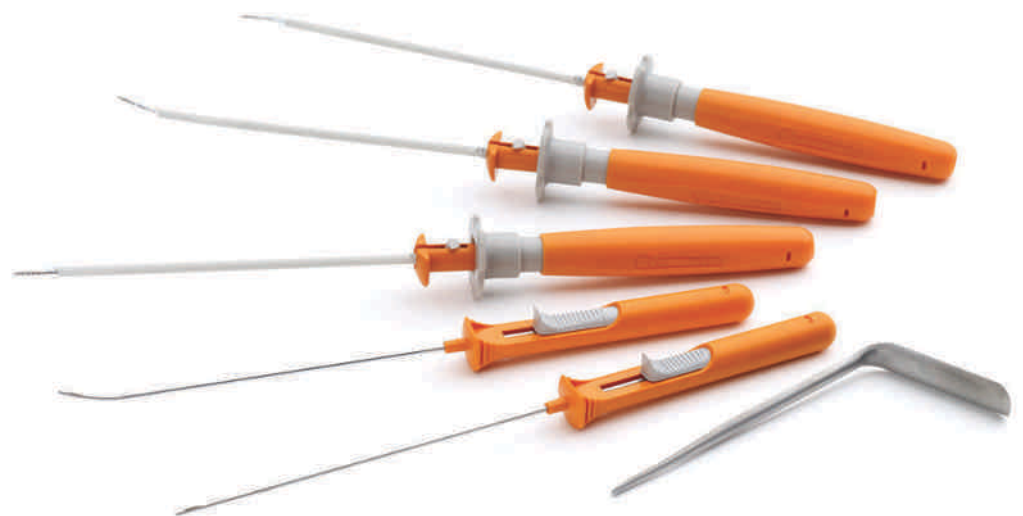

All-Inside Meniscal Repair with the FAST-FIX[®] 360 Meniscal Repair System

Introduction

The meniscus contributes to optimal knee joint function because of its critical role in load transmission, shock absorption, joint stability, lubrication, articular cartilage nutrition, and neuromuscular proprioception. Clinical studies have demonstrated that even partial meniscectomy can lead to early joint chondrosis and arthrosis¹. As a result, repair of peripheral and red/white meniscal tears is now the standard of care.

Due to the ability to predictably place vertical or horizontal mattress sutures on the femoral or tibial surfaces of the meniscus, the inside-out repair technique is considered by many surgeons to be the “gold standard” for meniscal repair. However, the inside-out technique requires a posterior incision and dissection to avoid neurovascular complications, thereby adding morbidity and operative time to the procedure. The technique also requires a trained assistant to retrieve and tie the repair sutures.

The outside-in repair technique was introduced in an attempt to eliminate the need for a posterior incision and dissection. However, the outside-in technique offers limited access to tears in the posterior third of the meniscus and limits the surgeon’s ability to perform a vertical mattress suture and place sutures on the tibial surface of the meniscus.

The all-inside arthroscopic suture-based technique has addressed many of the limitations of the inside-out and outside-in techniques. The all-inside suture-based technique has gained popularity because of the following advantages:

- The repair can be safely performed without a posterior incision.
- The technique allows easy access to tears in the posterior and middle thirds of the meniscus.
- The repair can be performed without a trained assistant.
- The technique allows vertical or horizontal mattress sutures to be inserted on the femoral or tibial surface of the meniscus.

These advantages allow for minimally invasive meniscal repair, resulting in less postoperative pain and morbidity for patients².

As described by:

Charles H. Brown, Jr., MD
Medical Director
Abu Dhabi Knee & Sports Medicine Centre
Abu Dhabi, United Arab Emirates

Nicholas Sgaglione, MD
Associate Chairman and Program Director
Department of Orthopaedic Surgery
North Shore-Long Island Jewish Medical Center
New Hyde Park, New York

Overview

The FAST-FIX[®] 360 Meniscal Repair System provides a strong, reproducible, and time-saving technique for meniscal repair. Biomechanical studies have demonstrated that a vertical mattress suture performed using the FAST-FIX 360 Meniscal Repair System has biomechanical properties equal to that of a vertical mattress suture performed using the open and inside-out repair techniques. This result can be achieved without the previously mentioned disadvantages and limitations of the inside-out and outside-in repair techniques, and without the need for intra-articular knot tying. The system gives the surgeon the versatility of placing horizontal or vertical mattress sutures on the femoral or tibial surfaces of the meniscus, potentially minimizing risk to the posterior neurovascular structures.

Building on the proven clinical success of the earlier FAST-FIX and ULTRA FAST-FIX Meniscal Repair Systems' techniques, the all new FAST-FIX 360 Meniscal Repair System offers the following advantages:

- Easier and faster implant deployment with added safety features
- More controlled implant delivery
- Smaller insertion points, minimizing disruption to the meniscus
- Less needle advancement needed for implant delivery
- A built-in depth penetration limiter
- Stiffer needle shaft for enhanced delivery control
- Ability to reposition the needle for optimal suture placement

As a result, the FAST-FIX 360 Meniscal Repair System (Figure 1) helps optimize the chances of a successful meniscal repair.

As with all arthroscopic procedures, adequate joint distention and visualization of the meniscus tear are essential for success. To minimize the potential for damage to neurovascular structures, it is strongly recommended that the surgeon use the built-in, adjustable depth penetration limiter to control the depth of penetration of the delivery needle. The tear is precisely assessed and the point of needle insertion from the peripheral rim is measured with a meniscal depth probe. This measurement is used to set the built-in adjustable depth penetration limiter.

Unlike the earlier FAST-FIX Meniscal Repair System, implants in the new FAST-FIX 360 Meniscal Repair System are pushed out of the delivery needle rather than being stripped away from the needle. This new delivery method minimizes the depth of needle penetration needed to successfully deploy the implants, thereby decreasing the risk of injury to nearby neurovascular structures. The FAST-FIX 360 Meniscal Repair System also employs a new ergonomically shaped 360° deployment slider which makes a “clicking” sound upon deployment of the implants. These new features provide a more controlled method to ensure proper implant deployment.

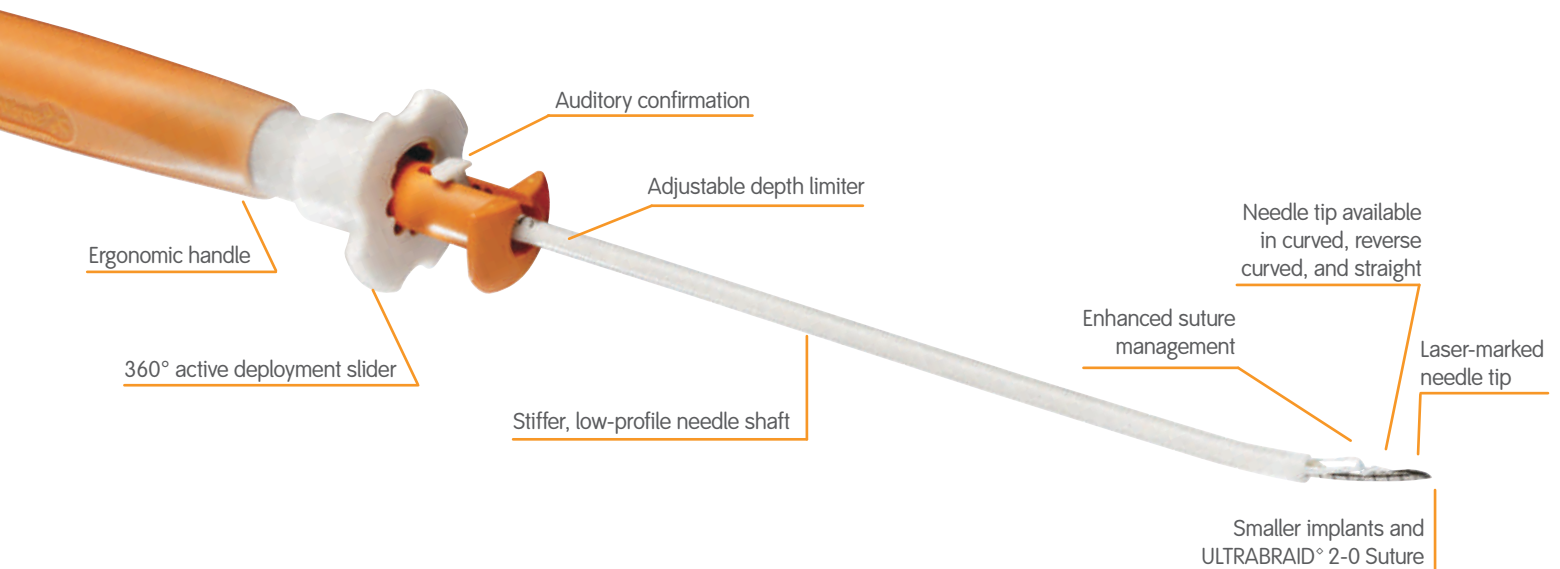


Figure 1. The FAST-FIX 360 Meniscal Repair System.

Setup

Each FAST-FIX[®] 360 Meniscal Repair System contains two 5 mm polymer integrated bio-inert anchors (PEEK-OPTIMA[®] from Invivo[®]) with a pretied, self-sliding knot comprised of 2-0, non-absorbable, UHMW polyethylene ULTRABRAID[®] Suture. The entire system is packaged in an easy-to-insert integrated delivery needle. The delivery needles are available in curved, straight, and reverse curved designs (Figure 2). The curved and reverse curved designs allow the surgeon to rotate the needle tip away from the neurovascular structures when penetrating the meniscus, further reducing the risk of neurovascular injury. The curved delivery needle is optimally shaped to allow vertical mattress sutures to be inserted on either the femoral or tibial surfaces of the meniscus. The reverse curved delivery needle is most useful for repairing tears on the tibial surface and more anterior located tears. The anchors are placed into the meniscus sequentially and are seated safely outside the capsule. The suture is then tensioned in a simple manner without the need for arthroscopic knot tying.

The built-in, adjustable depth penetration limiter is adjustable from 10 mm to 18 mm from the tip of the needle. The device comes with the depth penetration limiter preset at 18 mm. Patients with small knees, peripheral tears in the red-red zone, tears around the popliteus hiatus, and tears in the middle third of the medial meniscus may require a delivery depth less than the preset depth of 18 mm. Use of the meniscal depth probe in conjunction with the adjustable depth penetration limiter (white plastic sheath) allows controlled delivery of the implants.



Figure 2. Delivery needle designs.

Procedure Setup and Portal Placement

The operating room setup includes a lateral thigh post or leg holder to allow application of valgus stress to the knee to open the medial or lateral compartments for easier access to the tear. Use a surgical skin marker to outline the following surface landmarks: the patella, the medial and lateral borders of the patellar tendon, and the medial and lateral joint lines. Create the anterolateral portal at the level of the inferior pole of the patella as close as possible to the lateral border of the patellar tendon. Perform diagnostic arthroscopy, identify the meniscal tear, and assess its suitability for repair. Create the anteromedial portal under direct arthroscopic visualization. Insert an 18 gauge needle through the skin above the medial joint line. Adjust the needle position to allow optimal insertion of the FAST-FIX 360 delivery needle. Adjust the external starting position for the needle so that the needle can be placed perpendicular to the tear. Create the anteromedial portal in routine fashion using a #11 blade. It is important to dilate the portal with the arthroscopic blunt obturator to allow for easier passage of the delivery needle into the joint.

Repair lateral meniscal tears by viewing the tear through the anterolateral portal and using the anteromedial portal for the delivery needle. For a medial meniscal tear, use the meniscal depth probe to determine if the sutures should be inserted through the anteromedial portal or if it is necessary to switch the scope to the anteromedial portal and insert the sutures through the anterolateral portal. In general, medial meniscal tears in the middle third zone are best repaired by inserting the sutures through the anterolateral portal. This approach allows the sutures to be inserted perpendicular to the tear. Because the tibial spines can interfere with optimal suture placement when the sutures are inserted through the anterolateral portal, in some cases it may be necessary to insert the sutures through the anteromedial portal.

Procedure

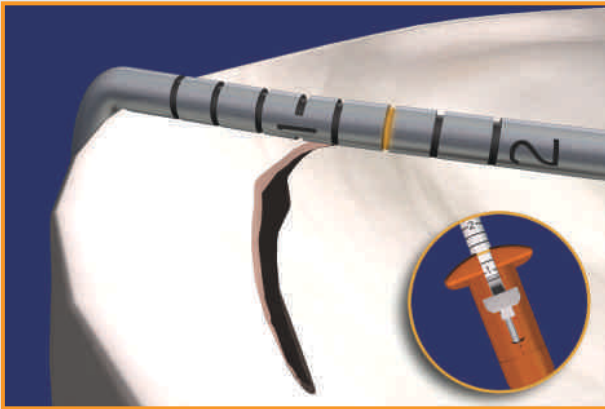


Figure 3. Establish depth limit using the adjustable depth limiter and depth probe.

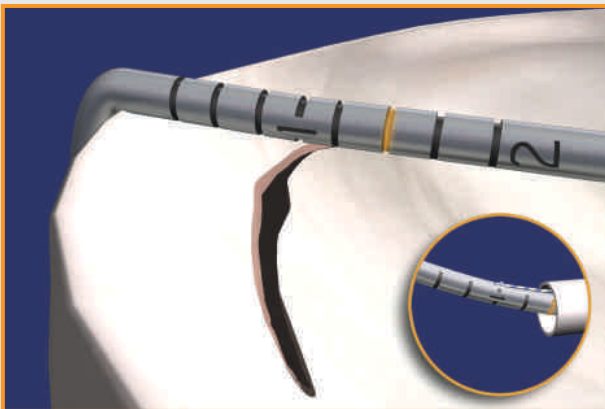


Figure 4. Use laser marks as a reference.

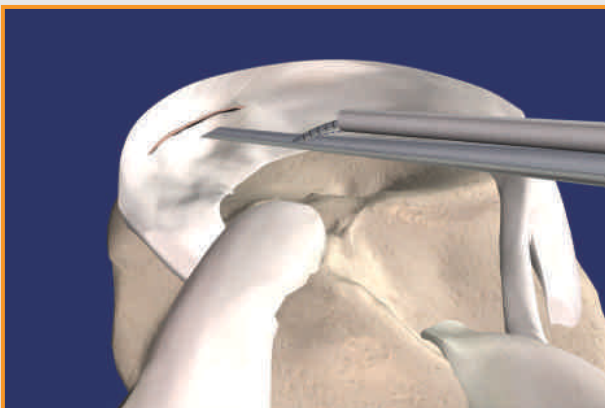


Figure 6. Introduce the delivery needle into the joint with the tip down against the slotted cannula.

Meniscal tear site preparation is essential for biological healing of the tear. Meniscal rasps and/or arthroscopic shavers are used to abrade and excoriate both sides of the tear and the perimeniscal synovium. Once the optimal portal placement is determined and the meniscal tear site is prepared, perform the repair as follows:

1. Use the meniscal depth probe to determine the desired depth limit. Place the tip of the probe at the meniscosynovial junction and measure the width of the meniscus at the desired entry point for the delivery needle. In the average size knee a depth of 14 mm is usually adequate. Adjust the depth penetration limiter to the desired length by pressing the depth limiter button (**Figure 3**). This length can be adjusted outside or inside of the joint. The laser marks on the tip of the needle can also be used as a reference (**Figure 4**).
2. Insert the FAST-FIX[®] 360 delivery needle into the joint through the appropriate arthroscopic portal. Insertion is facilitated through the use of the slotted cannula (sold separately) (**Figure 5**).



Figure 5. Use a slotted cannula to ease insertion.

The slotted cannula eases passage through the fat pad, and the cannula can also be used to help position the tip of the delivery needle at the desired location on the meniscus. Introduce the delivery needle through the slotted cannula into the joint, ensuring that the tip of the needle is pointing down (**Figure 6**). Once the needle is inside the joint, the slotted cannula may be removed if desired.

Pearl: Hold the delivery needle at the handle and push the slider with the thumb to deploy the implants. Do not advance the deployment slider while introducing the delivery needle into the joint or the implant will deploy prematurely.



Figure 7. Insert the delivery needle into the meniscus through the capsule.

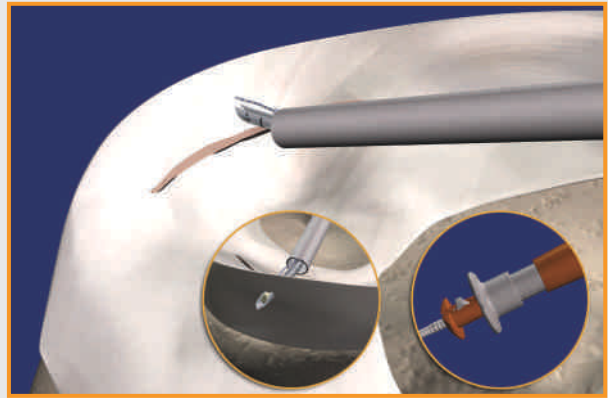


Figure 8. Keep the delivery needle in position during deployment of the implants.

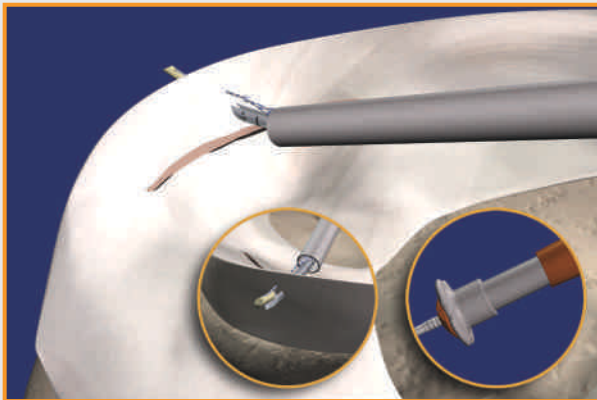


Figure 9. Push the deployment slider all the way forward to deploy T1.



Figure 10. Withdraw the delivery needle from the meniscus slowly for better suture management.

Vertical Mattress Suture Repair

- For a vertical mattress suture repair, place the first implant (T1) on the capsular side of the tear. Insert the FAST-FIX[®] 360 delivery needle into the capsule or into any remaining meniscal tissue on the capsular side of the tear (**Figure 7**). Use the slotted cannula to stabilize the meniscus, enhance visualization, and minimize skiving of the delivery needle to ensure more accurate placement of the implants. Position the tip of the slotted cannula at the desired entry point and rotate the cannula away from the direction of the neurovascular structures. Rotating the cannula allows better visualization of the delivery needle tip and directs the needle away from the neurovascular structures. Keeping the delivery needle in position, push the deployment slider all the way forward to deploy T1 (Figures 8 and 9). Proper deployment of the implant is accompanied by a “clicking” sound. For better suture management and to prevent pulling out the second implant (T2), release the deployment slider and slowly withdraw the needle out of the meniscus, keeping the needle inside the slotted cannula (if desired) and within arthroscopic view (**Figure 10**).

Pearl: Release the slider right after deployment of T1 to allow the “spring back” of the slider to its original position flush with the handle to pick up the T2 implant (**Figure 11**). Do not slowly release or hold the slider. If the slider does not spring back, the user may manually return the slider to its original position.



Figure 11. Make sure the trigger is flush to the handle prior to deploying T2.

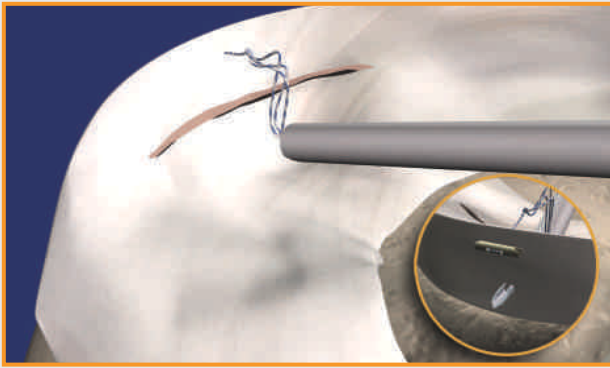


Figure 12. Advance the delivery needle to the preset needle depth limit.

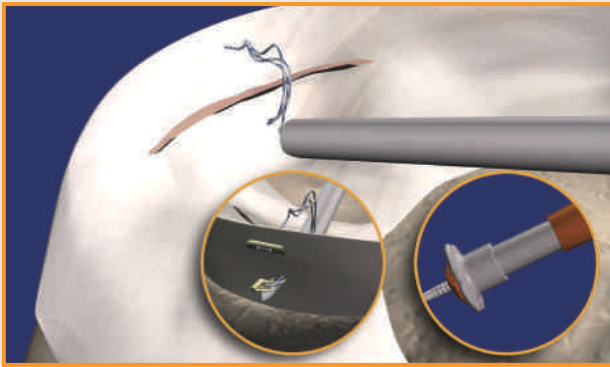


Figure 13. Push the deployment slider all the way forward to deploy T2.

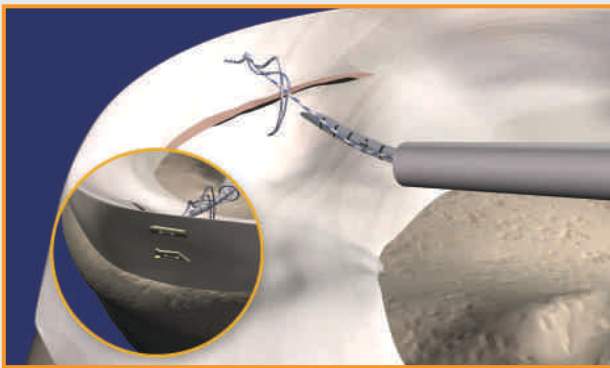


Figure 14. Withdraw the needle from the joint.

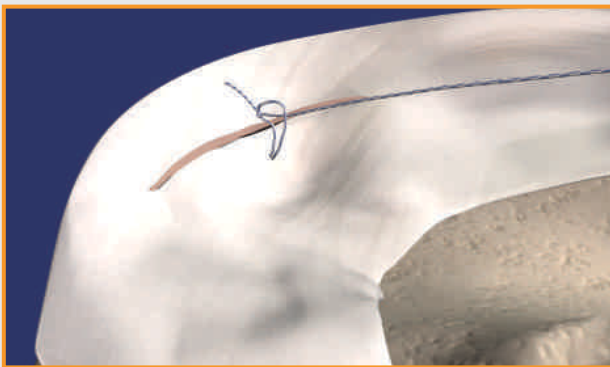


Figure 15. Pull the free end of the suture.



Figure 16. Apply tension to the suture to cinch the knot down.

Position the slotted cannula at the desired entry point on the inner meniscal fragment (if desired). The entry point for the second (T2) implant should be at least 5 mm from the tear site. Advance the delivery needle until the depth penetration limiter contacts the surface of the meniscus (**Figure 12**). Keeping the delivery needle in position, push the deployment slider all the way forward to deploy T2 (**Figure 13**). As with T1, proper deployment of T2 is accompanied by a “clicking” sound. Slowly withdraw the delivery needle from the joint after deployment of T2 (**Figure 14**).

Pearl: Do not push the deployment slider until the needle is fully penetrated through the meniscus to the preset depth limit or T2 will deploy prematurely.

Horizontal Mattress Suture Repair

- For a horizontal mattress suture repair, place the first implant (T1) at the posterior location. Place the delivery needle perpendicular to the tear and a minimum of 5 mm from the tear site on the inner meniscal fragment. Advance the delivery needle until the depth penetration limiter contacts the surface of the meniscus. Keeping the delivery needle in position, push the deployment slider all the way forward to deploy T1. Proper deployment of the implant is accompanied by a “clicking” sound. Release the deployment slider and slowly withdraw the delivery needle out of the meniscus, keeping the needle within arthroscopic view. Position the delivery needle more anteriorly along the meniscal tear site for the insertion of the second limb of the horizontal mattress suture. In general, maintain a minimum width of 8 mm between the two insertion points. Advance the delivery needle until the depth penetration limiter contacts the surface of the meniscus. Keeping the delivery needle in position, push the deployment slider all the way forward to deploy T2. As with T1, proper deployment of T2 is accompanied by a “clicking” sound. Slowly withdraw the delivery needle from the joint after deployment of T2.

Pearl: Do not push the deployment slider until the needle is fully penetrated through the meniscus to the preset depth limit or T2 will deploy prematurely.

- Remove the delivery needle from the knee, pulling the free end of the suture out of the joint. The free end of the suture is pulled to advance the sliding knot and reduce the meniscal tear (**Figure 15**). It is normal to encounter firm resistance as the knot is snugged down. It is important to pull the free end of the suture directly perpendicular to the tear site. Wrap the suture around several fingers and use the tibia as a fulcrum to provide a controlled method of tightening the knot. Slowly and steadily apply tension to the suture. In most cases, this steady pulling of the suture will cinch the knot down (**Figure 16**).

Pearl: If too much resistance is encountered while advancing the knot, use the Smith & Nephew Straight or Curved Knot Pusher/Suture Cutter (sold separately) to help facilitate removing suture laxity.

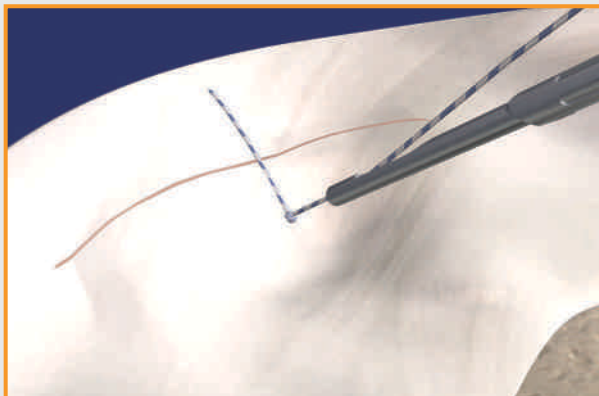


Figure 17. Slide the knot pusher/suture cutter to the knot.

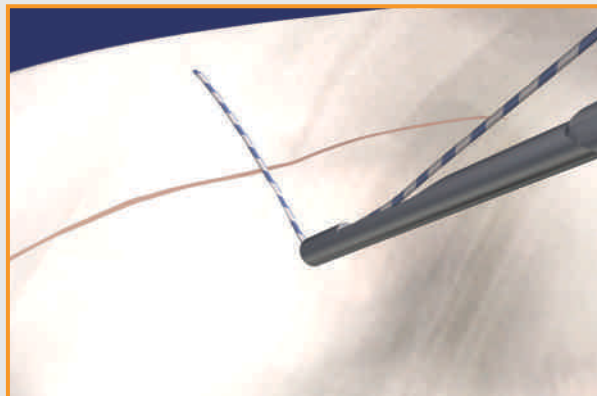


Figure 18. Push the knot pusher/suture cutter tip against the knot to recess the knot.

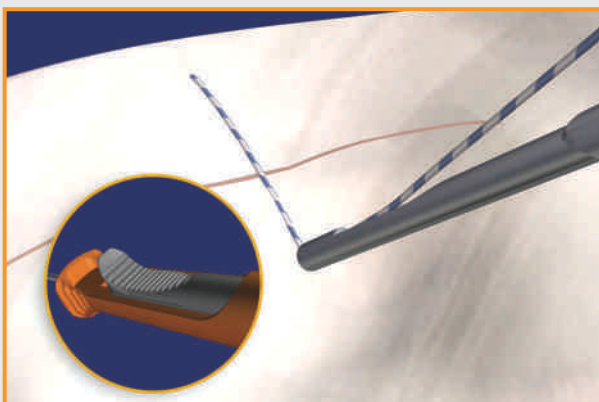


Figure 19. Push the trigger to cut the suture.

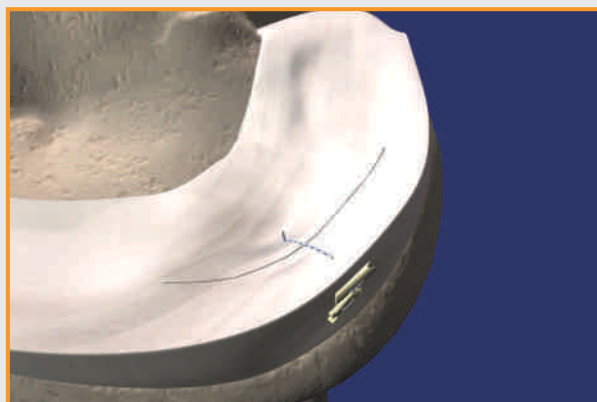


Figure 20. Completed vertical mattress stitch.

6. To further tighten the knot and further compress the tear site, thread the free end of the suture through the knot pusher/suture cutter. Both curved and straight knot pushers/suture cutters are available. Use a suture funnel to facilitate threading of the suture.
7. While holding the suture taut, gently slide the knot pusher/suture cutter to the knot (**Figure 17**). The knot pusher should engage the suture in a direct line perpendicular to the repair. A manual suture “pull”/“push” maneuver is suggested, and the knot should be tightened until the desired amount of compression is generated at the tear site.
8. Position the tip of the knot pusher/suture cutter against the knot to ensure a 2–3 mm suture tail when the suture is cut. Continuing to hold the suture taut, push the knot pusher/suture cutter tip against the knot. In some cases it is possible to recess the knot into the surface of the meniscus, leaving the tail of the suture flush with the surface of the meniscus (**Figure 18**). Cut the suture by pushing the trigger forward (**Figures 19 and 20**). Because of the high strength of the suture, using a small arthroscopic basket punch or scissors to cut the suture often results in the tail of the suture being frayed.
9. Place sutures on the tibial side of the tear as well as the femoral side of the tear to reduce puckering of the meniscus. The reverse curved delivery needle is especially useful for placing sutures on the tibial side of the tear.

Postoperative Care

The FAST-FIX[®] 360 Meniscal Repair System utilizes a high strength non-absorbable suture and allows the repair to be performed with a vertical mattress suture, which has been shown to be the strongest meniscal repair technique. As a result, the standard rehabilitation protocol used with inside-out repairs can be utilized.

Additional Instruction

Prior to performing this technique, consult the Instruction for Use documentation provided with individual components – including indications, contraindications, warnings, cautions, and instructions.

Pearls

- Prepare meniscal tear site properly.
- Choose the portal which most easily allows the delivery needle to be inserted perpendicular to the tear site.
- Set the depth penetration limiter.
- Use curved or reverse curved delivery needles.
- Vertical mattress suture: T1 inserted into the capsular side of the tear; T2 inserted on the meniscal side of the tear.
- Hold the device at the handle and push the slider with the thumb to deploy T1 and T2.
- Keep the device in position during deployment of T1 and T2.
- Release slider after deployment of T1 to allow “spring back” of the slider to pick up T2.
- For better suture management and to prevent pulling out T2, release the deployment slider and slowly withdraw the needle out of the meniscus.
- Thread suture onto the knot pusher/suture cutter with the suture funnel.
- If the knot does not cinch smoothly, it usually requires a steady and more forceful pull, which is facilitated by wrapping the suture around several fingers, like a pulley, and applying tension.
- Cinch the knot to obtain the desired degree of compression at the tear site.
- Avoid over-cinching the knot, which can result in puckering of the meniscus or the suture cutting through the meniscus and weakening the repair.
- Alternate divergent femoral side and tibial (tensile) side suture placement optimizes the strength of the repair and helps achieve an anatomic repair.
- Consider the reverse curved delivery needle for placing sutures on the tibial surface of the meniscus.
- Place the FAST-FIX 360 delivery needle either through the inferior (tibial) or superior (femoral) surface of the meniscus for optimal strength.
- For the easiest knot sliding and avoidance of the neurovascular bundles, insert the needle perpendicular to the tear using a contralateral approach. Use portals placed adjacent to the patella tendon to facilitate this procedure.
- The pretied, self-sliding knot included in the FAST-FIX 360 Meniscal Repair System slides from (T1) to (T2). Therefore, placing T1 further away than T2 facilitates sliding of the knot.
- Maintaining the needle insertion tip within the arthroscopic view at all times avoids potential suture tangling.

Ordering Information

To order the instruments used in this technique, call +1 800 343 5717 in the U.S. or contact your authorized Smith & Nephew representative.

| REF | Description |
|----------|--|
| 72202467 | FAST-FIX [®] 360 Straight Needle |
| 72202468 | FAST-FIX 360 Curved Needle |
| 72202469 | FAST-FIX 360 Reverse Curved Needle |
| 72202674 | Straight knot pusher/suture cutter and slotted cannula set, single use |
| 72202675 | Curved knot pusher/suture cutter and slotted cannula set, single use |

Accessories:

| | |
|---------|--------------------------------------|
| 015186 | Meniscal depth probe, reusable |
| 014549 | 45° Diamond rasp, reusable |
| 014550 | 90° Diamond rasp, reusable |
| 7210977 | Slotted cannula, reusable |
| 7210450 | Suture funnel, sterile, box of 10 |
| 7209950 | Suture threaders, sterile, box of 10 |

References

1. Ak. Joy Singh, Nilachandra L, Y.Nandabir Singh, Brogen Ak. Rehabilitation Following Arthroscopic Partial Meniscectomy - A Neglected Issue. *IJPMR* 15, April 2004; 1-6.
2. Nicholas A. Sgaglione, Meniscus Repair: Update on New Techniques: *Techniques in Knee Surgery* 1(2): 113-127, December 2002.

CAUTION: U.S. Federal law restricts these devices to sale by or on the order of a physician.

Courtesy of Smith & Nephew, Inc., Endoscopy Division

°Trademarks of Smith & Nephew, registered U.S. Patent & Trademark Office.
All trademarks acknowledged.



Endoscopy
Smith & Nephew, Inc.
Andover, MA 01810
USA

www.smith-nephew.com
+1 978 749 1000
+1 978 749 1108 Fax
+1 800 343 5717 U.S. Customer Service

©2010 Smith & Nephew, Inc.
All rights reserved.

04/2010 10600542 Rev. A